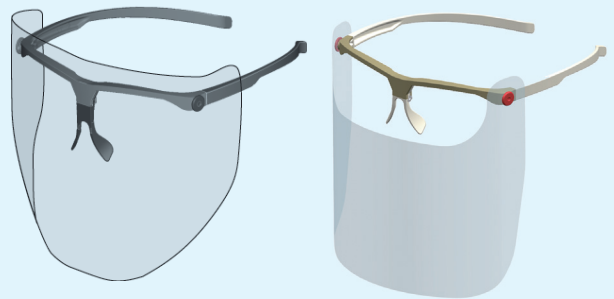
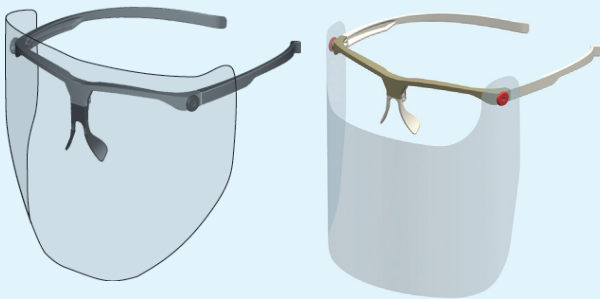
Mænnershield a brief manual

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

Assembly

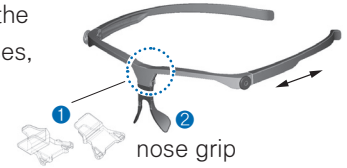
- Attach the ① adaptor to the goggles if you wear glasses, the ② nose grip if you do not.

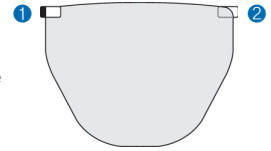
- Ensure the side with the colored (blue or green) protection film is facing your face on the inside. Remove the ① colored-protection film on the anti-fog side and then the ② clear protection film on the other side.

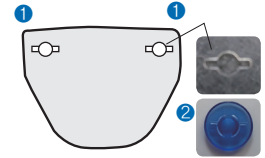- Align the horizontal marks on each side ① of the film and on the inside of the film hooks ②, fix them to the goggles on each leg. This allows you to move the shield film up and down your face

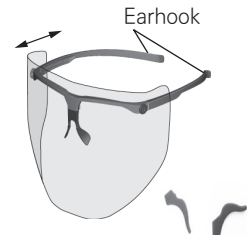- Use the ear hooks as necessary in order to hold the goggles in place

Additional shield films for refills can be purchased online at <https://store.coupang.com/link/marketinsight>

Components

1. Goggles

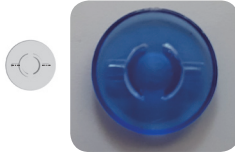
2. Nose grip

3. Adaptor

4. Film Hook


5. Ear Hook


6. Shield Film

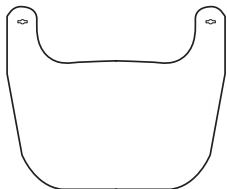Sight-Open Design

Full-Face Design

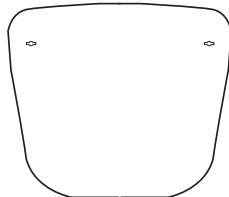Product Summary

Maennershield was designed to provide easy-to-wear every day protection against droplets and sprays for the general public. The goggles were constructed using an eco-friendly material called Ecozen by SK Chemical. The shield film was manufactured using Ecozen and optical PET

Product Features

Sight-Open Design to reduce strain on your eyes

Two different types of shield design allow users to select the most appropriate design as needed. For use over a long period of time, users can alternate between the Full-Face Design that covers the entire face and the Sight-Open Design which offers more comfort for your eyes

Anti-fog coating

Our anti-fog coating technology protects the film from fogging, securing clear visibility. protection against fogging from temperature difference, we also offer films with anti-fog coating on both sides.

Customized design for glasses wearers

Our nose grips were designed to be detachable for better fitting on top of your glasses. You may also use the adaptor for better grip

Anti-slip ear hooks

Ear hooks are provided so you can adjust the goggles to fit your face and head more firmly

Ergonomical light-weight design

Ergonomical, light-weight design allows to wear the goggles with comfort even for a long duration. (Full-Face Design: 24g, Sight-Open Design: 18g)